



Quality Management System Certificate No. 1058  
AS/NZS ISO 9001:2008

**Sexton Trading Company Pty Ltd**

ABN 68 009 829 485

72 Crockford Street Northgate Qld 4013 Australia

PO Box 135 Hamilton Qld 4007 Australia

**Telephone: (61 7) 3265 9999**

Facsimile: (61 7) 3260 6079

Web: [www.sexton.com.au](http://www.sexton.com.au)

Email: [sexton@sexton.com.au](mailto:sexton@sexton.com.au)

## ***Pressure Relief Mattresses***

### ***CARE INFORMATION***

---

The top and bottom covers of the pressure relief mattresses should be cleaned with soap, water and a soft bristle brush ***before*** wipe down / disinfection procedures to prevent staining then use standard cleaning solutions.

These solutions must be properly diluted according to manufacturer's instructions using regular hospital wipe down / disinfection procedures.

The following cleaning agents can be used on the pressure reduction mattress system without fabric or coating breakdown or loss of waterproofing!

8% Chlorine Boil  
10% Chlorine Wipe down  
Quaternary Wipe down (Properly Diluted)  
Phenolic Wipe down (Properly Diluted)  
Soap and Water

Other cleaning guidelines:

- The family of low pressure mattresses must be cleaned after each patient use.
- Staff should wear appropriate protective apparel when cleaning mattresses.
- Laundering is ***not*** recommended and is not necessary under normal circumstances.
- After wipe down / disinfection procedures have been completed, allow the top cover surface to completely air dry before bed makeup.
- If the top cover retains an unpleasant odour, a final wipe down with a scented mouthwash (less than 4% alcohol) will freshen it. Lemon scented cleaners disinfectants may also be used.

The zippered cover does not need to be removed for cleaning, nor should it be opened during use. The zipper is provided to allow repair or replacement of component parts, if required. Should this be necessary, please contact your sales representative.



Quality Management System Certificate No. 1058  
 AS/NZS ISO 9001:2008  
**Sexton Trading Company Pty Ltd**  
 ABN 68 009 829 485  
 72 Crockford Street Northgate Qld 4013 Australia  
 PO Box 135 Hamilton Qld 4007 Australia  
**Telephone: (61 7) 3265 9999**  
 Facsimile: (61 7) 3260 6079  
 Web: [www.sexton.com.au](http://www.sexton.com.au)  
 Email: [sexton@sexton.com.au](mailto:sexton@sexton.com.au)

## ***Cleaning Solutions and Disinfectants***

In general the following products, when used in those concentrations recommended by the manufacturer, may be used satisfactorily.

Iodophor type disinfectants are not recommended for use because staining may result.

TRADE NAME	DISINFECTANT TYPE	MANUFACTURERS	*MANUFACTURERS RECOMMENDED DILUTIONS
A 33	Quaternary	Airwick (Professional Products Div.)	2 oz./gal.
A 33 (dry)	Quaternary	Airwick (Professional Products Div.)	½ oz./gal.
Beaucoup	Phenolic	Ecolab	1 oz./gal.
Blue Chip	Quaternary	S. C. Johnson	2 oz./gal.
Elimstaph	Quaternary	Walter G Legge	1 oz./gal.
Franklin Phenomysan F2500	Phenolic	Purex Corp.	1 ½ oz./gal.
Franklin Sentinel	Quaternary	Purex Corp.	2 oz./gal.
Galahad	Phenolic	Puritan/Churchill Chemical Co.	1 oz./gal.
Hi-Tor	Quaternary	Ecolab	½ oz./gal.
LPH	Phenolic	Vestal Laboratories	½ oz./gal.
Matar	Phenolic	Ecolab	½ oz./gal.
Omega	Quaternary	Airwick (Professional Products Div.)	½ oz./gal.
Quanto	Quaternary	Ecolab	1 oz./gal.
Sanikleen	Quaternary	West Chemical Products	2 oz./gal.
Sanimaster II	Quaternary	Service Master	1 oz./gal.

This list is not intended to be all-inclusive nor should it be considered a recommendation or endorsement of any product. Dilute as specified on the product label. Disinfectants applied at full concentration or in highly concentrated solutions may eventually cause problems and prove harmful to the fabrics.